


Preloaded system Accuject™ Pro



**DIVE DEEP INTO
THE LUXSMART™
MOVEMENT**
Discover peer and patient testimonials and how LuxSmart™ can help you.

Loading Guide for Preloaded IOL Injector Accuject™ Pro with LUXSMART™, LUXSMART™ Crystal, LuxGood™ and LuxGood Crystal™

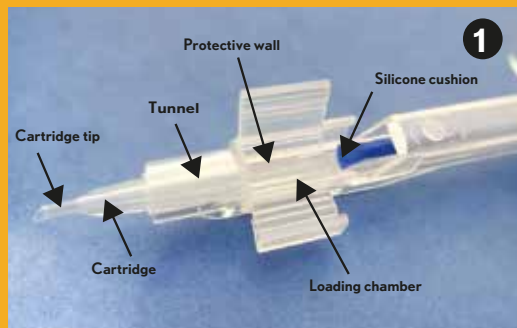

The Accuject™ Pro is a push-type fully preloaded single use injection system



Open the blister and transfer the injection system onto the sterile field.



Rinse the lens by slowly injecting BSS* at room temperature from the tip of the cartridge inclined upwards.



Then, rinse the lens from the back of the cartridge inclined downwards by positioning the cannula near the silicone tip under the protective wall. The cartridge contains a coating that is activated when hydrated by the addition of BSS*. The coating fixes a layer of fluid that ensures lubrication. Full hydration of all the parts is mandatory.



Apply a little amount of viscoelastic solution** in the tip of the cartridge to retain the BSS inside the cartridge and the tunnel.



Wait for 60 seconds




Close the wings of the cartridge until you hear a "click". The system is now ready for injection (do not advance the lens into the tunnel yet), please now is the moment to give it to the surgeon.



For the implantation, remove the excess viscoelastic by advancing the plunger slowly and continuously until it reaches halfway along the tunnel.



Check that the trailing haptics are not covering the optic. Then, insert the cartridge tip into the incision and slowly and continuously push the plunger until the lens is delivered. It is very important not to stop during the pushing process.

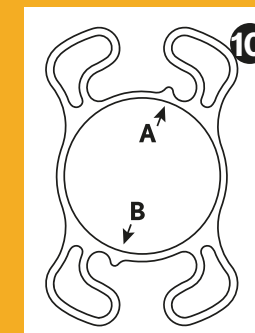

Once implanted, the orientation features of the lens must be oriented at the top right (A) and at the bottom left (B)

@BauschSurgical
Bausch + Lomb Surgical

www.luxsmartiol.com
www.bauschsurgical.eu

*BSS: Balanced Salt Solution
**Amvisc® Plus, OcuCoat® and EyeFill® C.